Unboxing


Waveband 1 box contents

Waveband (Model WB1CLMD)

Components to be used in combination with the Waveband 1.0 Headband :


- Power Supply Unit (Ref: PN 00019)
- Adjustment strips : (Ref: PN 00093 for L, PN 00092 for M, PN 00091 for S, PN 00090 for XS)
- Travel Bag: (Ref: PN 00096)

Documentation: Quick Start Manual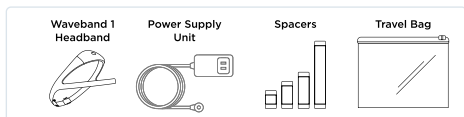

Getting Started


Download the Beacon Pal app on your mobile device by going to beacon.bio/setup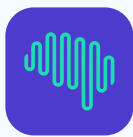

- Ensure your mobile device is connected to the internet.
- Ensure your mobile device Bluetooth is turned on.
- Ensure you have the credentials provided to you.

Ensure your device has enough battery to start a recording. A full charge may take up to 2 hours.

Note: When the headband is fully charged the LED will be **solid green**. If the headband is not fully charged, plug it in as shown below.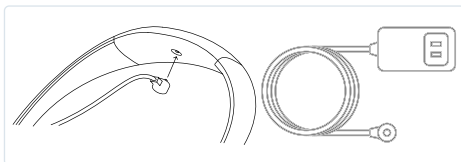

You can record with, or without the mobile app.

For further information, please refer to the *Instructions For Use* available in the Beacon Pal Settings tab.

Start Recording with Mobile App

1. Open the Beacon Pal app.

2. Activate your account (if applicable). If your account is not already activated, enter your activation credentials.

Your mobile device Bluetooth must be on, and connected to the internet.

3. Setup the headband by following the steps in the Beacon Pal app. Pair the headband to the mobile device over Bluetooth. Follow the steps to update the headband if instructed by the app.

4. Follow the recording setup steps in Beacon Pal to put on the headband correctly and get ready to start recording.


- Ensure the mobile device's Bluetooth remains on while recording.
- Keep the mobile device charging nearby while recording.

5. When ready, start recording using the Beacon Pal App. Do not stop the recording to readjust the headband. The headband will emit a 3-tone audio cue to indicate the recording started.


Start Recording without Mobile App

1. Turn on the headband by pressing the power button. The LED will turn white while the system is booting. The LED will return to green when the headband is ready.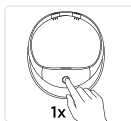

If the LED is ruby or blinking ruby, charge the headband. When the LED is green, the headband has sufficient battery to record.

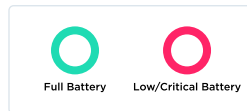2. Put on the headband and get ready to start recording.

a) For long hair: tie up hair, put on headband and brush away hairs caught in the device.

b) Put on the headband: make sure the front sensors are touching the forehead and the back sensors are touching of the scalp.

c) The headband should be snug yet comfortable. The headband should not budge while it is on your head.

d) If the headband is too tight: add size adjusters until the device is snug yet comfortable.

Note: A good fit means good data quality!

3. When ready, start recording by pressing the power button 3 times in rapid succession. The headband will emit a 3-tone audio cue to indicate the recording started. The LED will turn off shortly after the recording is started.

Wear the headband the entire time a recording is in progress.

Recordings cannot be paused. Only stop a recording when you are ready.

Ending a Recording

Ending a recording with the Mobile App

1. Stop recording in the Beacon Pal app at the end of a recording session. Once a recording is stopped, it cannot be restarted.

2. Follow the post-recording steps in Beacon Pal to ensure the recording is uploaded successfully. The mobile device's Bluetooth must be turned on and connected to the internet. Check the upload status of recording data in the Beacon Pal app.

Ending a recording without the Mobile App

If you are not using a companion app to record, stop recording by holding down the power button for 3 seconds. The headband will emit a 3-tone audio cue to indicate the recording ended.

3. Plug in the headband to charge. Verify the charging cable is connected to the magnetic charging port.

Recording Steps Completed.

Headband Indicators

Turning on headband				4 beeps
Start a recording				3 beeps
Recording in progress				No sound
Stop a recording				3 beeps
Turning off headband				4 beeps
Battery too low to record				3 beeps
Memory too low to record				3 beeps

- Bluetooth pairing in progress
- Firmware update in progress
- General Error
- Pulsing: Memory almost full; Solid: Memory full.

Headband Memory Limitations

The device has sufficient on-device storage for approximately 50 hours of total recording time. Connect headband to mobile app to retrieve recordings, and free up memory. Without use of mobile app, device must be returned.

Symbols

	Type BF Applied Part
	Temperature Limitation
	Fragile, handle with care
	Follow Instructions for Use
	Do not throw in trash. Requires a distinct disposal process.
	Caution, Consult Accompanying Documents
	Protect from heat and radioactive sources
	Pressure limitation
	Humidity limitation
	Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Degree of protection against ingress of water and particulate matter (IPNIN2)
	Ingress Protection Rating (IP22) for POD only
	Date and Place of Manufacture (Taiwan)
	Manufacturer
	Unique Device Identifier
	Catalog number
	Serial number

Warnings, Conditions and Storage

Do not place on sores, open wounds or rashes. If any skin irritation occurs, stop use immediately. Use only as instructed. Keep device and accessories away from children and pets. Modification (or attempted modification) of device and accessories is forbidden and may be dangerous. Do not use device with accessories than the ones provided.

Gently clean the headband and power supply unit once a week with a dry cloth. Use a cotton swab with isopropyl alcohol (IPA) to remove any debris. Do not clean with detergents or cleaning gels.

Indoor use only. Do not expose to water. Use or storage outside these conditions may result in malfunction or premature wear.

Storage and transport: Temperature: +59°F to +95°F, +15°C to +35°C, Relative Humidity: 15% to 90% , Pressure: from 1060 hPa to 700 hPa

Environmental Operating Condition: Temperature: +59°F to +95°F, +15°C to +35°C, Relative Humidity: 15% to 90%, Pressure: from 1060 hPa to 700 hPa

Manufacturer: Beacon Biosignals, 22 Boston Wharf Road 7th Floor, Unit 41 Boston, MA 02210
Contact: support@beacon.bio

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